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Our reference: FSCA 2340

Dear Sir / Madam

**IMPORTANT : URGENT FIELD SAFETY NOTICE**  
**Toxo ISAGA IgA ref. 79322 lot 1003405320**

Our records indicate that your laboratory has received one or more kits of the following product:

Product Number	Description	Lot number
79322	Toxo ISAGA IgA	1003405320

### **Description of the issue**

An error occurred during the Manufacturing Process of Toxo ISAGA IgA kits (ref. 79322, lot 1003405320). Some strips coated with anti-human IgA monoclonal antibodies (R1) show a mistake in their labeling. The letter "A" which indicates that the strips detect anti-Toxoplasma IgA antibodies is missing. All of the strips in the pouch are missing the letter "A" indication.

### **Impact:**

If the TOXO ISAGA IgA kits (ref. 79322) are used in combination with TOXO ISAGA kits (ref. 73561 – used in the detection of Toxoplasma IgM antibodies), the potential risk associated with the product issue is the user will mistake an IgA strip for an IgM strip and report IgA results as IgM results. The highest risk among all the possibilities is the risk of IgM false negative results.

Cont....

**Required actions:**

We request you take the following actions at this time:

- Please distribute this information to all appropriate personnel in your laboratory, retain a copy in your files, and forward this information to all parties that may use this product, including others to whom you may have transferred our product.
- Discard any remaining inventory of the lot 1003405320 Toxo ISAGA IgA – ref. 79322 and complete and return the acknowledgement form for replacement.
- Discuss any concerns you may have regarding previously reported results with your Laboratory Medical Director to determine the appropriate course of action.
- Complete and return the attached acknowledgement form in ATTACHMENT A via fax for the attention of UK Vigilance Team to fax number : 0044 (0) 1256 461881 or scan and email to [UKVIGILANCE@biomerieux.com](mailto:UKVIGILANCE@biomerieux.com) to acknowledge receipt of this URGENT FIELD SAFETY NOTICE (customer letter)

We can advise that the Competent Authorities have been made aware of this notification..

bioMérieux is committed to providing our customers with the highest quality product possible. We sincerely apologise for any inconvenience that this may have caused you. If you require additional assistance or have any questions, please contact your local bioMérieux Customer Service representative.

Yours sincerely

Customer Service Department