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ANEXO

www.ecri.org . Printed from *Health Devices Alerts* on Thursday, April 19, 2018 Page 1

[Normal Priority] - S0343 : ZOLL—ZOLL M and R Series Defibrillators with End Tidal CO2 Adapter—May Require Zeroing When “Check CO2 Adapter” Message is Displayed [ECRI Exclusive User Experience Network]
Medical Device Special Report

Published: Thursday, March 22, 2018

UMDNS Terms:

- Defibrillators, External, Manual [11134]
- Defibrillator/Pacemakers, External [17882]

Product Identifier:
[Capital Equipment]

Product	ZOLL Medical Corp Model
Defibrillators with End Tidal CO2 (EtCO2) Option	M Series, R Series

Geographic Regions: Worldwide

Manufacturer(s): ZOLL Medical Corp 269 Mill Rd, Chelmsford, MA 01824-4105, United States

Suggested Distribution: Cardiology/Cardiac Catheterization Laboratory, Clinical/Biomedical Engineering, Critical Care, Emergency/Outpatient Services, Nursing, Risk Management/Continuous Quality Improvement

Problem:

1. When switching between a reusable and a disposable EtCO₂ adapter, or vice-versa, the ZOLL R & M Series defibrillators will display a “Check CO₂ Adapter” message.
2. Clinicians may not recognize that to address the message, they may need to zero the adapter.
3. When this problem occurs, confirmation of the correct airway placement and confirmation that the patient is receiving ventilation may be delayed.

ECRI Recommendations:

1. For ZOLL Series defibrillators that have the EtCO₂ option, make it standard practice to zero the defibrillator for the type of adapter connected if EtCO₂ use is anticipated.
2. Inform users to zero the adapter in the following scenarios:
 - when a “Check CO₂ Adapter” message is displayed
 - when switching between adapter types

Background:

1. A member reports that new ZOLL R Series EtCO₂ units displayed the “Check CO₂ Adapter” message instead of the EtCO₂ value.
 1. The units had been zeroed with a reusable adapter but were being used with a disposable adapter.
 2. Following zeroing with the currently connected adapter, the units displayed a valid EtCO₂ value.
2. The ZOLL instructions for use (IFU) states that when switching adapter types, it is recommended to zero the adapter.
3. Zeroing the adapter takes up to 20 seconds.
4. “Check CO₂ Adapter” message is also displayed if the adapter is not tight or if the adapter window has moisture or debris.

Comments:

- This alert is a living document and may be updated when ECRI Institute receives additional information. In circumstances in which we determine that it is appropriate for customers to repeat their review of an issue (e.g., when additional affected product has been identified), we will post a separate update alert. In other cases, we may add information, such as additional commentary, recommendations, and/or source documents, to the original alert. For additional information regarding the format of this alert, refer to our [HDA Format Guide](#).

Source(s):

- 2018 Feb 28. ECRI Institute member researched report